



Use

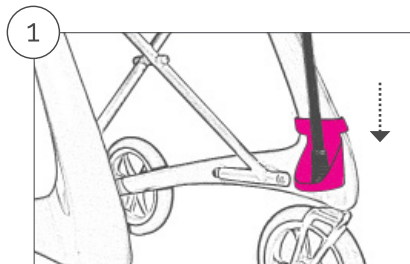

1. Place the bottom of the cane in the bottom part of the cane holder.
2. Then push the cane into the clip attached to the seat frame.

To detach the cane simply push the cane forward until it is released from the clip on the seat frame. Then lift the cane out from the bottom part of the cane holder.

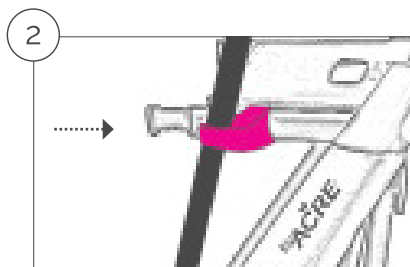

Technical specifications

Material: 3D Print

Height: 10 cm

Width: 6 cm

Diameter: 5,5 cm


Maintenance

The cane holder can be cleaned with a wet cloth. If necessary, it can also be cleaned with some washing-up liquid.

Cane holder

KEEPS YOUR CANE IN PLACE WHILE WALKING

CARBON ULTRALIGHT ACCESSORY

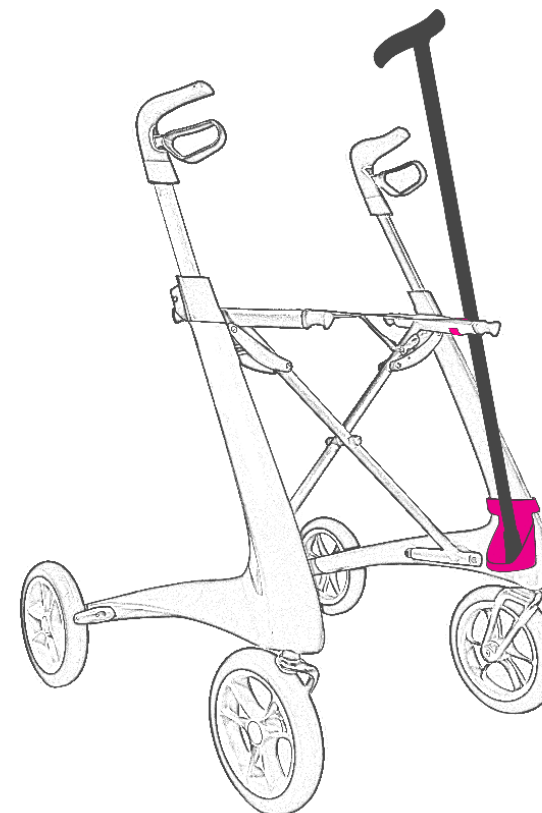

Congratulations on your new cane holder!
We have made this user manual for you, so that you can quickly enjoy your new, stylish cane holder.

by **ACRE**

Designed in Denmark. Handcrafted in China.
byACRE ApS, I.C. Modewegsvej 1, 2800 Lyngby

ENGLISH

by **ACRE**


Use of the cane holder

The cane holder is easy to attach and has a drain at the bottom, so that water and dirt are not collected.


The illustrations in this manual are based on the right-side cane holder.

If you have the cane holder for the left side you simply do the attachment and detachment on the left side instead.


Attachment

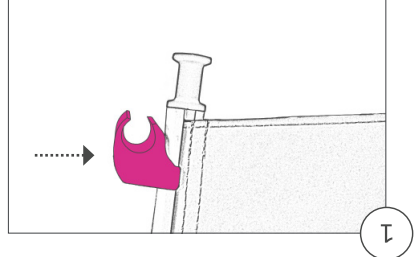

1. Click the small clip onto the seat frame in the side where your cane holder should be placed.

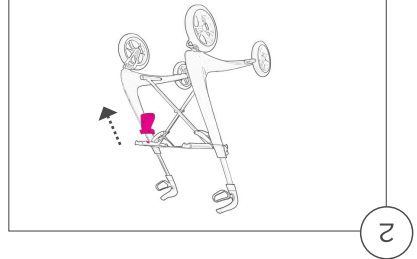


2. Push the bottom part of the cane holder onto the narrow part of the rollator frame on top and push it downwards...

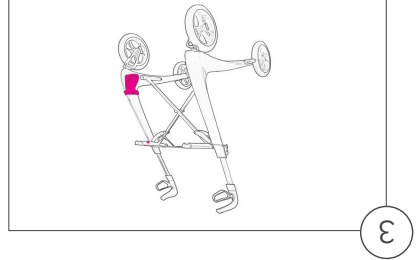


3. ... until it is placed on the widest part of the rollator frame.


Detachment

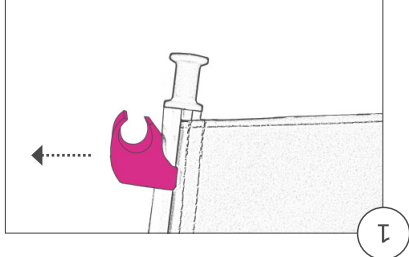

1. Gently pull the clip off the seat frame

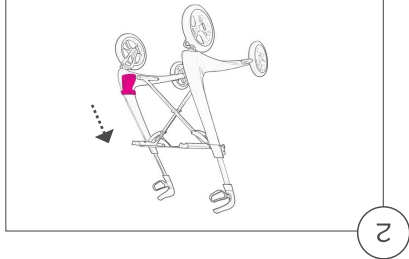

2. Gently pull the bottom part of the cane holder upwards along the rollator frame...

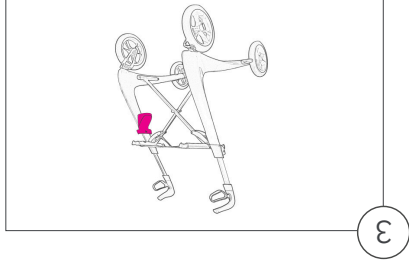

3. ... until it can be detached from the frame.